



CASE REPORT

Intense Pulsed Light - FT

SYNCHRO REPLA:Y
SYNCHRO FT
MINISILK FT
DOT 1540



The Code of Excellence

Synchro REPLA:Y
Synchro FT
Minisilk FT
DOT 1540

PIGMENTED LESIONS

Pigmented Lesions Treatment on Cheeks

48 years old female Patient
Phototype III

Settings

Fluence	19 J/cm ²
Filter	500 nm
Nr. of Pulses	2
Pulse Duration	2 ms + 6 ms
Delay	10 ms



Before treatment



After 2 sessions

Courtesy of:
G. Cesare, MD
Dermatologist
Naples, Italy

Synchro REPLA:Y
Synchro FT
Minisilk FT
DOT 1540

PIGMENTED LESIONS

Pigmented Lesions Treatment on Decolletè

35 years old female
Phototype III

Settings

Fluence **18 J/cm²**

Filter **550 nm**

Nr. of Pulses **2**

Pulse Duration **2 ms + 6 ms**

Delay **20 ms**



Before treatment



After 1 session

Courtesy of:
G. Cesare, MD
Dermatologist
Naples, Italy

Synchro REPLA:Y
Synchro FT
Minisilk FT
DOT 1540

PIGMENTED LESIONS

Hand Dyschromias

Caucasian female
Phototype II

Settings

Fluence 13-20 J/cm²

Filter 550 nm

Nr. of Pulses 2

Pulse Duration 4 ms + 6 ms

Delay 10 ms



Before treatment



One month after the 4th treatment

Courtesy of:
P. Campolmi, MD
G. Cannarozzo, MD
P. Bonan, MD
Department of Dermatology
University of Florence, Italy

Synchro REPLA:Y
Synchro FT
Minisilk FT
DOT 1540

PIGMENTED LESIONS

Hand Dyschromias

Caucasian female
Phototype II

Settings

Fluence 12-20 J/cm²

Filter 550 nm

Nr. of Pulse 2

Pulse Duration 4 ms + 6 ms

Delay 10 ms



Before treatment



One month after the 3rd treatment

Courtesy of:
P. Campolmi, MD
G. Cannarozzo, MD
P. Bonan, MD
Department of Dermatology
University of Florence, Italy

Synchro REPLA:Y
Synchro FT
Minisilk FT
DOT 1540

PIGMENTED LESIONS

Hand Dyschromias

Caucasian female
Phototype II

Settings

Fluence	12-13 J/cm ²
Filter	550 and 650 nm
Nr. of Pulses	2
Pulse Duration	3 ms
Delay	20 ms



Before treatment



One week after the 3rd treatment

Courtesy of:
A. Pulvirenti, MD
Aging Medicalaser
Catania, Italy

Hand Dyschromias

Caucasian female

Phototype II

Settings

Fluence 12-13 J/cm²

Filter 550 and 650 nm

Nr. of Pulses 2

Pulse Duration 3 ms

Delay 20 ms



Before treatment



One week after the 3rd treatment

Courtesy of:
A. Pulvirenti, MD
Aging Medicalaser
Catania, Italy

Synchro REPLA:Y
Synchro FT
Minisilk FT
DOT 1540

STRETCH MARKS

Stretch Marks Treatment

Left Buttock

25 years old female

Phototype III

Settings

Fluence 20 J/cm²

Filter 500 nm

Nr. of Pulses 2

Pulse Duration 6 ms

Delay 10 ms



Before treatment



After 2 sessions

Courtesy of:
G. Cesare, MD
Dermatologist
Naples, Italy

Synchro REPLA:Y
Synchro FT
Minisilk FT
DOT 1540

STRETCH MARKS

Stretch Marks Treatment Right Buttock

25 years old female
Phototype III

Settings

Fluence 20 J/cm²

Filter 500 nm

Nr. of Pulses 2

Pulse Duration 6 ms

Delay 10 ms



Before treatment



After 2 sessions

Courtesy of:
G. Cesare, MD
Dermatologist
Naples, Italy

Synchro REPLA:Y
Synchro FT
Minisilk FT
DOT 1540

VASCULAR LESIONS

Angioma on the Back

40 years old female
Phototype III

Settings

Fluence 21 J/cm²

Filter 500 nm


Nr. of Pulses 2

Pulse Duration 7 ms

Delay 10 ms



Before treatment

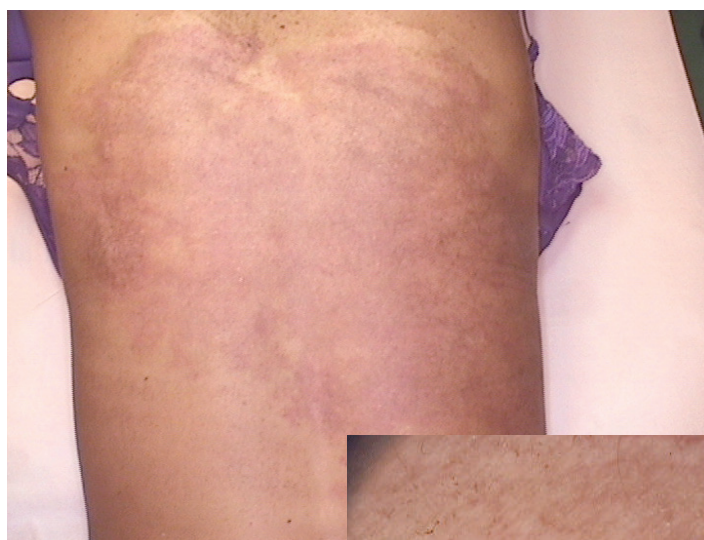

After 10 sessions every
30-40 days

Courtesy of:
G. Cesare, MD
Dermatologist
Naples, Italy

Synchro REPLA:Y
Synchro FT
Minisilk FT
DOT 1540

VASCULAR LESIONS

Pigmented Angiodermia on Lower Limbs

40-years old
Caucasian female
Phototype III

Settings

Fluence 16-20 J/cm²

Filter 550 nm

Nr. of Pulses 2

Pulse Duration 4 ms + 7 ms

Delay 10 ms



*Before the treatment
(picture taken using polarized light)*



*Five months after the 6th treatment
(picture taken using polarized light)*

Courtesy of:
P. Campolmi, MD
G. Cannarozzo, MD
P. Bonan, MD
Department of Dermatology
University of Florence, Italy

Synchro REPLA:Y
Synchro FT
Minisilk FT
DOT 1540


PHOTOREJUVENATION

Pigmented Facial Photorejuvenation

Caucasian female
Phototype III-IV

Settings

Fluence	10-12 J/cm ²
Filter	550 and 650 nm
Nr. of Pulses	2
Pulse Duration	3 ms + 4 ms
Delay	20 ms

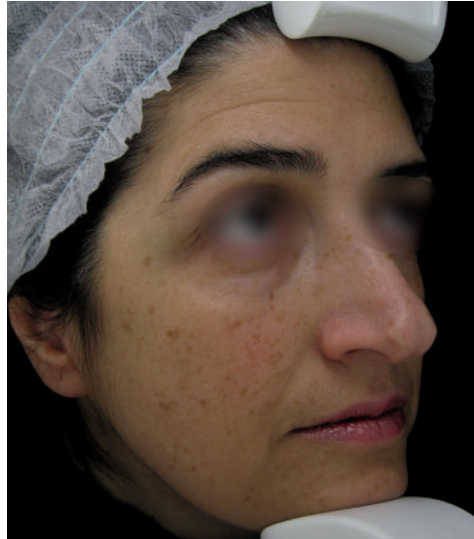

Before treatment



One week after the 3rd treatment

Courtesy of:
A. Pulvirenti, MD
Aging Medicalaser
Catania, Italy

INTENSE PULSED LIGHT - FT | CASE REPORT

Synchro REPLA:Y
Synchro FT
Minisilk FT
DOT 1540

PHOTOREJUVENATION

Pigmented Facial Photorejuvenation

Caucasian female
Phototype III-IV

Settings

Fluence	10-12 J/cm ²
Filter	550 and 650 nm
Nr. of Pulses	2
Pulse Duration	3 ms + 4 ms
Delay	20 ms



Before treatment



One week after the 3rd treatment

Courtesy of:
A. Pulvirenti, MD
Aging Medicalaser
Catania, Italy

INTENSE PULSED LIGHT - FT | CASE REPORT

Synchro REPLA:Y
Synchro FT
Minisilk FT
DOT 1540

PHOTOREJUVENATION

Pigmented Facial Photorejuvenation

Caucasian female
Phototype II

Settings

Fluence 10-13 J/cm²

Filter 550 nm

Nr. of Pulses 2

Pulse Duration 3 ms + 6 ms

Delay 20 ms



Before treatment



One week after the 2nd treatment

Courtesy of:
A. Pulvirenti, MD
Aging Medicalaser
Catania, Italy

INTENSE PULSED LIGHT - FT | CASE REPORT

Synchro REPLA:Y
Synchro FT
Minisilk FT
DOT 1540

PHOTOREJUVENATION

Pigmented Facial Photorejuvenation

Caucasian female
Phototype II

Settings

Fluence 10-13 J/cm²

Filter 550 nm

Nr. of Pulses 2

Pulse Duration 3 ms + 6 ms

Delay 20 ms



Before treatment



One week after the 2nd treatment

Courtesy of:
A. Pulvirenti, MD
Aging Medicalaser
Catania, Italy



The Code of Excellence

www.dekalaser.com

DEKA M.E.L.A. s.r.l.

Via Baldanzese,17 - 50041 Calenzano (FI) - Italy
Tel. +39 055 8874942 - Fax +39 0558832884