

PRODUCTS

onda

The Future of Body Sculpting

WITH SMART MICROWAVE TECHNOLOGY FOR VARYING BODY CONTOURING APPLICATIONS, THE ONDA DEVICE FROM DEKA COULD WELL BE THE FUTURE OF BODY SHAPING.

With impressive, long-lasting results, Onda is a game changer in body contouring and shaping. Made by DEKA in Italy and distributed in Australia by Aussie Medi Tech, Onda is Italian for 'wave', so named as it uses Coolwaves microwave energy, to deliver effective and extremely satisfying results for the patients.

‘THE FIRST DEVICE TO HARNESS MICROWAVE FREQUENCIES THAT TARGET DEEPER LAYERS, WHILE KEEPING THE SKIN COOL, ONDA TREATS FAT POCKETS, CELLULITE AND SKIN LAXITY IN ONE SIMPLE, COMFORTABLE TREATMENT.’

Onda Indications

1. Localised fat deposits

With a handpiece that allows Coolwaves to penetrate deeply, Onda is able to disrupt the membranes of the subcutaneous fat cells. The lysate is then drained off and eliminated by the body's natural metabolic processes. The integrated cooling system of the handpiece ensures that the skin remains cool and patient comfort is maximised.

2. Cellulite

Onda's shallow handpiece is able to effectively target the connective septa between the adipose globules of advanced cellulite, typically visible as an 'orange peel' effect. It can break this up and dispose of via the body's metabolic processes, while at the same time ensuring skin laxity is minimised.

3. Skin Tightening

Onda's microwaves trigger instant shrinkage of the collagen fibres within the dermis, thereby helping to firm and tone the tissue within the skin's deeper and more superficial layers. Three treatments are recommended for optimal results.



Dr Lee Lua

from B Clinic in Sunnybank, QLD, discusses his success with Onda

BREAKING NEW GROUND

The first device to harness microwave frequencies that target deeper layers, while keeping the skin cool, Onda treats fat pockets, cellulite and skin laxity in one simple, comfortable treatment.

Microwaves are part of the electromagnetic spectrum (frequency range: 1-300GHz). The unique design of Onda's smart Coolwaves handpieces mean that these microwaves are emitted precisely and accurately, in order to control the depth of treatment and ensure safer, more effective results.

SUPERIOR HANDPIECE

Unlike many radiofrequency systems, which have a much higher absorption by water molecules, the unique design of the Onda handpiece means that the microwaves are generated at a frequency that allows the majority of the thermal effect to be deposited in the subcutaneous fat. This results in more energy reaching the fat, a lower risk of excessive heating and reduced discomfort during treatment.

HEATING THINGS UP

What's more, Onda is able to heat the subcutaneous fat and connective tissue around eight to 10 degrees hotter than RF modalities are, while maintaining the same or lower skin temperature. Research suggests this can produce a greater reduction in fat cells in less treatment time. It also allows Onda to reach the desired temperature thresholds necessary for truly effective cellulite reduction and skin tightening. **AMP**

For more information call 1300 257 020 or visit www.amtlaser.com.au

What has patient satisfaction been like after using Onda?

We have received only positive feedback from our clients. Most of clients have seen noticeable improvement just after one session, but usually it takes a few weeks to see the results. We need to allow time for the natural lymphatic system to drain away the fat and cellular waste that is produced from the treatment. As the lymphatic system continuous to work, results will only improve, and new collagen continues to form.

Is there any downtime?

There is no downtime or recovery period. Patients can go back to their normal activities after each session.

Is the treatment painful?

The treatment is well tolerated by most of our clients. Some clients have even described it as a hot deep tissue massage.

How many treatments are required for best results?

Generally, we recommend three to four treatments for the best results, however some clients have seen immediate improvements even after one session.

How long do results typically last?

With a healthy diet and regular exercise results will last, as the damaged fat cells are completely removed from the lymphatic

system. For maintenance we also recommend our clients return every six months.

What other procedures would you use it as an adjunct to?

We can use the Onda for patients who have undergone liposuction, where they also require skin tightening and smoothing of the skin texture. We may use it for clients who have had body contouring treatments with other technologies elsewhere that have been left with some tissue irregularities. Onda helps to smooth and even out these irregularities.

What benefits has it brought to your practice?

Having the Onda has allowed us to meet the needs of a wide range of clientele and provide results faster than other devices on the market, which leads to our goal of high patient satisfaction.

Having a device that provides all the desired body shaping treatments in one platform means that we can treat a range of different client needs and we are not limited to one application.

It allows us to treat not only fatty deposit but also conditions like cellulite and skin laxity at different depths of action. This means that one client may want to have fat treatments and then return for skin tightening. Our clients who have undergone a liposuction procedure can also return to treat the areas that require some tightening.